INTENDED PURPOSE

For use in platelet aggregation studies to confirm specific platelet disorders.

WARNINGS AND PRECAUTIONS

The reagents contained in this kit are for In-Vitro diagnostic use only. **DO NOT INGEST.** Wear gloves when handling all kit components. Refer to the product safety data sheet for risk and safety phrases and disposal information. Plasma products have been screened and found negative (unless otherwise stated on the kit box or vial) for the presence of Hepatitis B Antigen (HbsAg) HIV 1 and 2 antibody and HCV antibody; however they should be handled with the same precautions as a human plasma sample.

COMPOSITION**1. Collagen Reagent 2x1.0ml**

Each vial contains 100µg of equine tendon collagen. The reagent should be a uniform clear suspension after gentle mixing. Any discolouration or uneven appearance may indicate contamination. Mix gently before use.

2. Other kit components

Each kit contains Instructions For Use

STORAGE AND SHELF LIFE

Unopened vials are stable until the given expiry date when stored under conditions indicated on the vial or kit label.

1. Collagen Reagent

The opened reagent is stable for 4 weeks when stored at 2...6°C.

SAMPLE COLLECTION AND PREPARATION

Plastic or siliconised glass should be used throughout. Blood (9 parts) should be collected into 3.2% or 3.8% sodium citrate anticoagulant (1 part). Separate platelet-rich plasma after centrifugation at 150-200 x g for 10-15 minutes (18...25°C). Separate platelet-poor plasma after centrifugation at 2000 - 3000 x g for 15 minutes. Dilute the platelet-rich plasma with platelet-poor plasma to obtain a platelet count of c 250 x 10⁹/l. Plasma should be kept at 18...25°C. Testing should be completed within 2 hours of sample collection^{1,2}.

STEP-BY-STEP PROCEDURE

No dilution is necessary prior to use, although if dilution is required it should be performed in saline and used immediately.

QUALITY CONTROL

Each laboratory should establish a quality control program. Normal and abnormal controls should be tested prior to each batch of patient samples to ensure satisfactory instrument and operator performance. If controls do not perform as expected, patient results should be considered invalid.

REFERENCE VALUES

Reference values can vary between laboratories depending on the techniques and systems in use. For this reason each laboratory should establish its own normal range.

PERFORMANCE CHARACTERISTICS

Within run CV's for Helena aggregation reagents using the PACKS-4 were <5%

BIBLIOGRAPHY

1. National Committee for Clinical Laboratory Standards (NCCLS): Collection, Transport and Preparation of blood specimens for coagulation testing and performance of coagulation assays. Document H21-A2:Vol II, No. 23, 1991.
2. Dacie, J & Lewis, S.M. 'Practical Haematology' 6th Ed. 1984: 250-251.

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